



## American Medical Manufacturing, Inc.

Mission Industrial Park

8741 Shirley Avenue, Northridge, CA 91324 USA • Tel: (818) 701-7180 • Fax to Email (818) 350-5738 • Fax: (818) 701-9708  
Website: www.amiwelisten.com • E-Mail: Sales@amiwelisten.com

### I.V. POLE & IRRIGATION TOWER PLUNGER/BRACKET ASSEMBLY P/N 55-BRK

#### INSTALLATION INSTRUCTIONS

1. Unscrew the sheet-metal screw (1) from the tube adjacent to the plunger hole.\*

\*If no screw exists, drill 1/8" diameter hole as shown in 1 below. Place the bracket on the tube and attach with a new stainless steel screw enclosed. (Do not tighten completely yet).

2. Drill 1/8" diameter hole through the bracket hole and the tube as shown in 2 below. Fasten the 2<sup>nd</sup> enclosed sheet-metal screw through the hole and tighten both screws until snug.

3. The Irrigation Tower is now ready for use.

